

## Så avslutar du programmet GLUCOFACETS DELUXE och tar ut din CONTOUR® USB på ett säkert sätt:

1. Högerklicka på ikonen till GLUCOFACETS DELUXE. Du finner den nederst till höger på skärmen.



2. En rullmeny visas – välj avsluta

A screenshot of the Glucofacts DELUXE software interface. The window title is 'Bayers GLUCOFACETS Deluxe versjon V2.01.12'. The main area displays 'CONTOUR USB: Serienr.7390-1065590' and a data table for the period '14. januar 2010 til 27. januar 2010'. The table has columns for 'Morgen 00:00 - 08:00', 'Middag 08:00 - 16:00', and 'Kveld 16:00 - 00:00', each with sub-columns for 'Ingen merking', 'Apple', and 'Gj.sn.'. A 'DATAANALYSE' section is visible below the table. On the right, there is a 'TEGNFORKLARING' (Legend) section. A context menu is open over the 'Avslutt' (Exit) icon in the top right corner, with the option 'Avslutt' selected.

Dato	Morgen 00:00 - 08:00			Middag 08:00 - 16:00			Kveld 16:00 - 00:00		
	Ingen merking	Apple	Gj.sn.	Ingen merking	Apple	Gj.sn.	Ingen merking	Apple	Gj.sn.
torsdag 14.jan.2010									
fredag 15.jan.2010									
lørdag 16.jan.2010									
søndag 17.jan.2010									
mandag 18.jan.2010									
tirsdag 19.jan.2010									
<b>DATAANALYSE</b>									
Ant. målinger				3	1		1		
Standard Avvik				6,3					
Gj.sn.				10,8	3,8		23,8		
Laveste måling				3,6	3,8		23,8		
Høyeste måling				15,4	3,8		23,8		

3. Nu kan du högerklicka på på Safely Remove Hardware ikonen och välja avsluta.

The screenshot shows a Microsoft Word window titled 'Document1 - Microsoft Word' with a Glucofacts DELUXE software interface. The software displays data for 'CONTOUR USB' (Serienr. 7390-1065590) for the period 14 January 2010 to 27 January 2010. The data is organized into columns for 'Morgen' (06:00 - 08:00), 'Middag' (08:00 - 16:00), and 'Kveld' (16:00 - 02:00). A 'SATAANALYSE' table is also present, showing statistics for 'Ant. målinger', 'Standard Avvik', 'Gj.sn', 'Laveste måling', and 'Høyeste måling'.


	Morgen 06:00 - 08:00	Middag 08:00 - 16:00	Kveld 16:00 - 02:00
Dato	Ingen merking	Ingen merking	Ingen merking
14 jan. 2010			
15 jan. 2010			
16 jan. 2010			
17 jan. 2010			
18 jan. 2010			


	Morgen 06:00 - 08:00	Middag 08:00 - 16:00	Kveld 16:00 - 02:00
Ant. målinger	3	1	1
Standard Avvik	6,3		
Gj.sn	16,8	3,8	73,8
Laveste måling	3,6	3,8	73,8
Høyeste måling	16,4	3,8	73,8

The Windows taskbar at the bottom shows the 'Safely Remove Hardware' icon highlighted, indicating the next step in the process.

#### 4. Denna ruta visas – välj Stop.

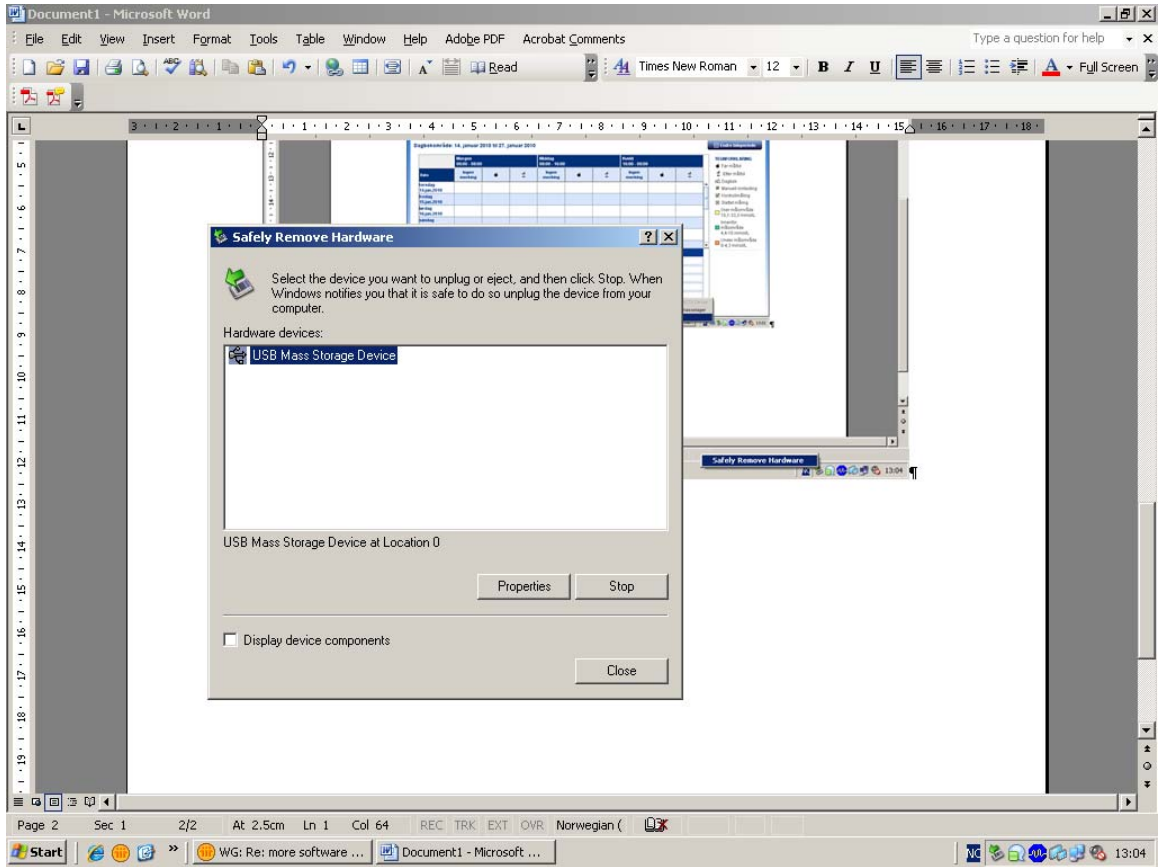

## 5. Välj "OK"

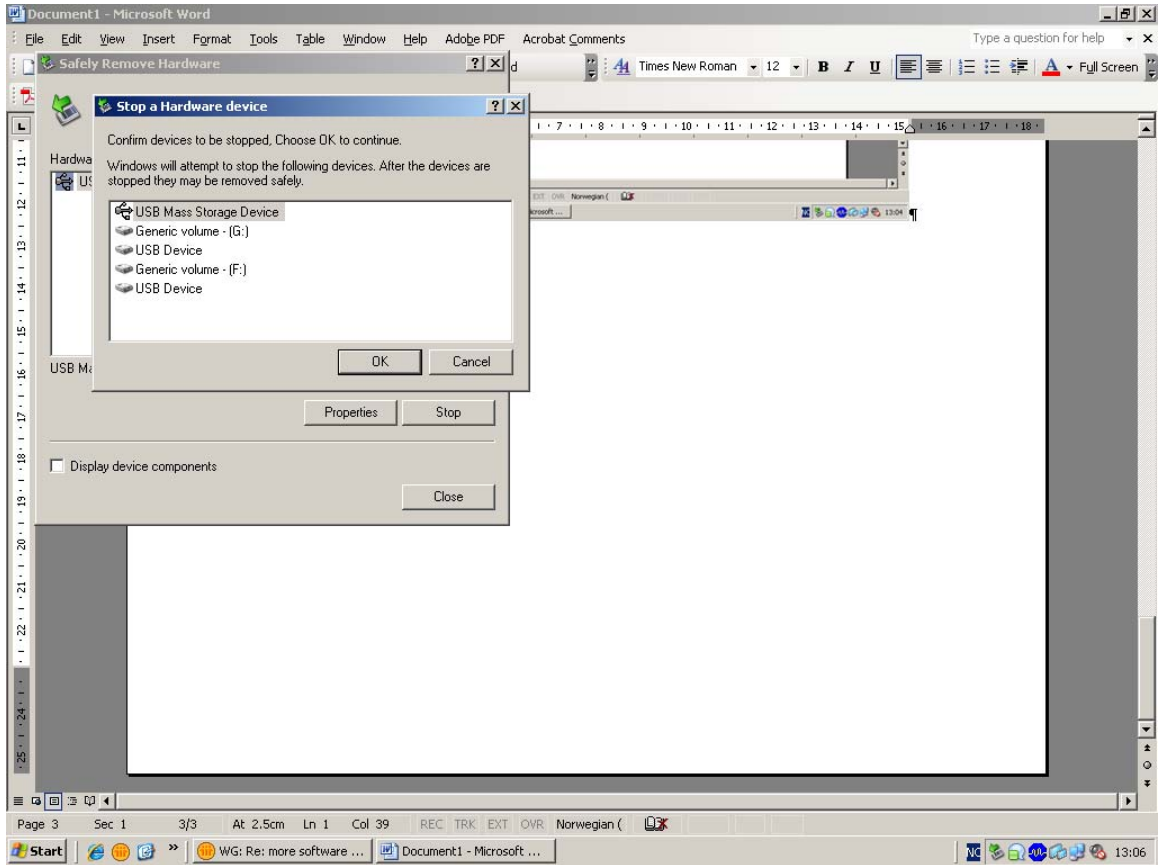

## 6. Du kan nu ta ut din CONTOUR USB.

